

SYSTEM COMPONENTS:

414-00-300 - NEXA® DEVICE KIT

- NEXA DEVICE
- NEXA CARRY CASE
- NEXA POWER SUPPLY with region specific adaptor
- INSTRUCTIONS FOR USE
- QUICK START GUIDE

414-00-501 - NEXA® DRESSING -125mmHg (BOX OF 5)

414-00-511 - NEXA® DRESSING -75mmHg (BOX OF 5)

- NEXA BLACK RETICULATED FOAM 150mm x 75mm x 20mm
- NEXA PRESSURE CONTROL CONNECTOR, DRESSING PAD AND TUBING
- POLYURETHANE DRAPE

414-00-502 - NEXA® FLUID CONTAINER PACK (BOX OF 5)

- 120ML FLUID CONTAINER & NEXA® DYNAMIC FLUID PUMP AND TUBING

UK DISTRIBUTOR
REGEN
Medical
PART OF THE JOINT OPERATIONS FAMILY

Unit 11, Interface Park, Bincknoll Lane,
Royal Wootton Bassett, Wiltshire,
SN4 8SY
Tel: +44(0)1793 575050
Email: customerservice@regenmedical.co.uk
www.regenmedical.co.uk

NEXA
MEDICAL

Nexa Medical Limited
Suite 5-6 Willow View, Southfields,
Church Lane, Bournemouth, BH22 8TR. UK
www.nexamedical.com
info@nexamedical.com

414-10-001 (E) 02/2020



NEXA®

THE NEXADVANTAGE FOR
NEGATIVE PRESSURE WOUND THERAPY

REGEN
Medical
PART OF THE JOINT OPERATIONS FAMILY

NEXA®
MEDICAL



OVERVIEW

The NEXA Negative Pressure Wound Therapy (NPWT) system has been developed to provide traditional, clinically proven, negative pressure therapy for patients with chronic or acute wounds.

With three new innovations in NPWT, NEXA offers more opportunity to transition patients from hospital to the community and reduce the risk of further re-admission due to complications with their wounds.

WHAT IS THE NEXADVANTAGE?

Conventional NPWT devices use a vacuum air pump to generate negative pressure which means exudate removal is dependent on the leak rate of the dressing.

NEXA utilises a Dynamic Fluid Pump which improves exudate removal and provides consistent negative pressure thanks to the novel Pressure Control Connector located close to the wound. The performance of NEXA is also not affected by height differences between the device and the wound. Exudate is drawn into a flexible bag, entirely separated from the pump mechanism.



The NEXA DEVICE

- One button operation for simplicity and ease of use.
- 8 week operational life and approved for multiple patient use.
- 10 hours battery life and 4 hour recharge time.
- Lightweight and small, suited to mobile patients.
- Carry case supplied to enable mobile use of the system.
- 'Days remaining' feature.



The NEXA Dynamic Fluid Pump

is constantly active, removing exudate from the wound irrespective of vacuum levels, thereby reducing the potential for skin maceration.



The NEXA FLUID CONTAINER PACK

- 120ml approximate volume.
- NEXA Dynamic Fluid Pump.
- Long tube length of 1.5 metres when connected to the NEXA DRESSING



The NEXA DRESSING (-75mmHg or -125mmHg)

- Sterile, single use dressing set.
- **NEXA Pressure Control Connector**, maintaining constant negative pressure, determined by the dressing selection.
- **NEXA Black Reticulated Foam**, attributed to effective negative pressure wound healing, as supported by the clinical evidence.